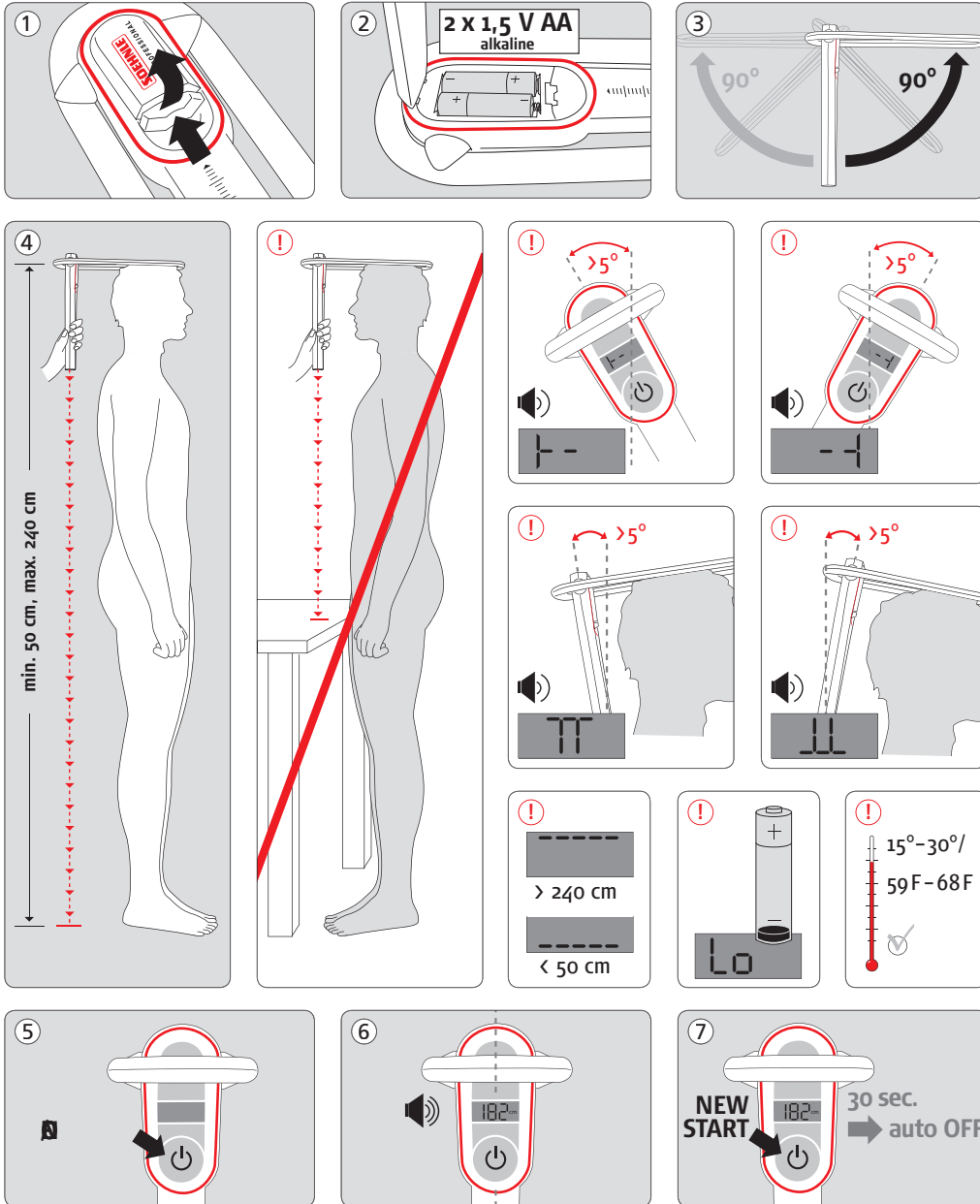
# LÄNGENMESSSTAB 5003

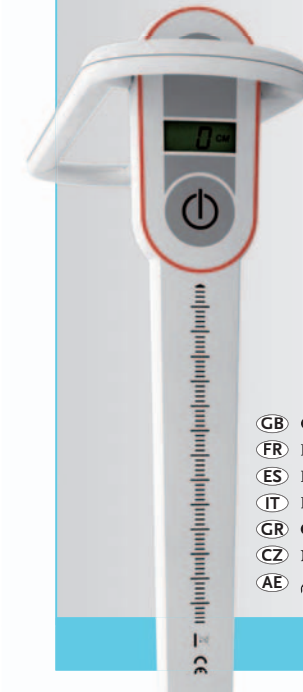
INBETRIEBNAHME + ANWENDUNG · GETTING STARTED + APPLICATION · MISE EN SERVICE + UTILISATION · PUESTA EN SERVICIO + APLICACIÓN  
ATTIVAZIONE E APPLICAZIONE · ΘΕΣΗ ΣΕ ΛΕΙΤΟΥΡΓΙΑ + ΧΡΗΣΗ · ÚVEDENÍ DO PROVOZU + POUŽITÍ · بداية التشغيل

# LÄNGEN- MESSSTAB 5003

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- GB OPERATING INSTRUCTIONS
- FR MODE D'EMPLOI
- ES MANUAL DE INSTRUCCIONES
- IT ISTRUZIONI PER L'USO
- GR ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
- CZ NÁVOD K OBSLUZE
- AE كيفية الاستخدام

## LÄNGENMESSSTAB 5003

WICHTIGE INFORMATIONEN · IMPORTANT INFORMATION · INFORMATIONS IMPORTANTES · INFORMACIÓN IMPORTANTE · IMPORTANTI INFORMAZIONI · ΣΗΜΑΝΤΙΚΕΣ ΠΛΗΡΟΦΟΡΙΕΣ · DŮLEŽITÉ INFORMACE · معلومات مهمة

### DE LÄNGENMESSSTAB 5003

#### Reinigung

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### GB HEIGHT ROD 5003

#### Cleaning

It is sufficient to clean the height rod using a damp cloth and a normal cleaning agent. Do not use any scouring agents.

#### Note

Interference suppression of the unit corresponds to the valid EU Directive 2004/108/EC. The display value may be influenced by extreme electrostatic and electromagnetic interference, e.g. operation of a radio unit or a mobile phone in the immediate vicinity of the unit. When electrostatic interference is no longer present, the appliance can again be used for its intended purpose. You may need to switch it on again. The appliance is a measuring instrument. Air currents, vibrations, rapid changes in temperature and direct sunlight may have an effect on the accuracy of the reading.

#### Warranty

The period of warranty shall be two years starting from the date of purchase. Please retain your invoice as proof of purchase. If your height rod requires servicing, please contact your dealer or Soehnle Professional Customer Service.

#### Disposal

According to the present state of the art the height rod contains no special environmentally harmful substances. This product is not to be treated as regular household waste, but should be handed in to an electrical/electronic equipment recycling centre. You can obtain further details from your local council, your municipal waste disposal company or the firm from which you purchased the product.

**Battery disposal:** Do not dispose of batteries in the household rubbish. As consumer you are obliged by law to return waste batteries. You can hand in your old batteries to the public collection centres in your municipality or return them to any store where batteries of the same type are sold. You will find these signs on batteries containing hazardous substances: Pb = Battery contains lead, Cd = battery contains cadmium, Hg = battery contains mercury.

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